



MULTI -PANEL DRUG TEST DIP CARD PROCEDURE SHEET

REVIEW INSTRUCTIONS.
TEST DEVICE, PATIENT'S SAMPLE, AND CONTROLS SHOULD BE BROUGHT TO ROOM TEMPERATURE 68° - 86°F (20° - 30°) PRIOR TO TESTING.
DO NOT OPEN POUCH UNTIL READY TO PERFORM THE ASSAY.
DOUBLE CHECK THE EXPIRATION DATE. DO NOT USE IF THE PRODUCT IS EXPIRED.

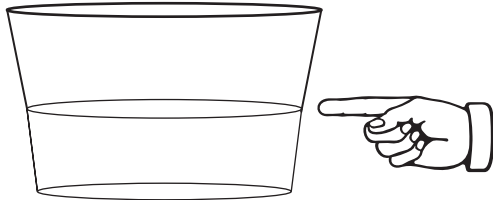


STEP 1
COLLECT SPECIMEN


COLLECT URINE SPECIMEN IN CLEAN CONTAINER.
RECOMMENDED LEVEL OF URINE IS 30 ML.

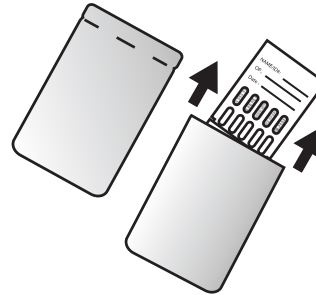

REMOVE FROM PACKAGE

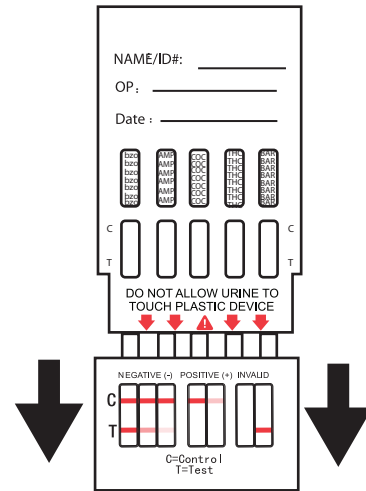


STEP 2
REMOVE CAP
REMOVE CAP FROM THE DIP CARD.

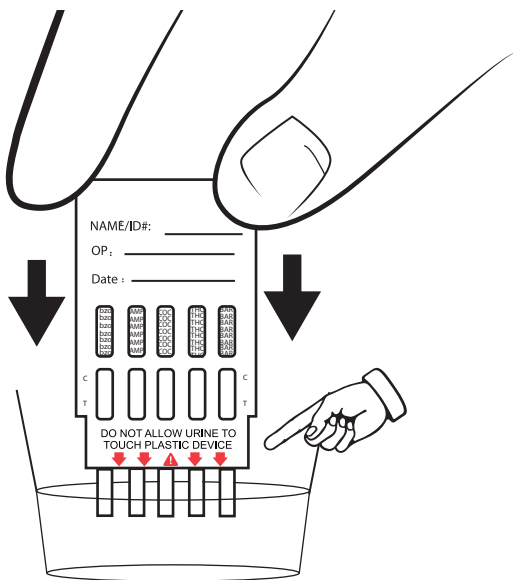



STEP 3
IMMERSE DIP CARD

IMMERSE POINTED END OF TEST STRIP IN CONTAINER OF URINE.
DO NOT TOUCH PLASTIC WITH URINE.
LEAVE IN FOR 20-30 SECONDS.

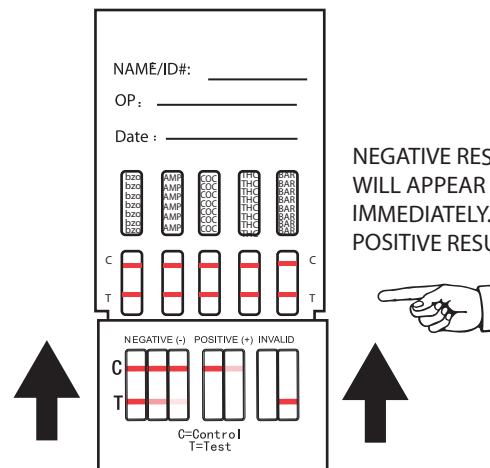

STEP 4
REMOVE DIP CARD
REMOVE DIP CARD AND REPLACE CAP.
PLACE FLAT ON DRY,
NON-ABSORBENT SURFACE AND WAIT FOR RESULTS.

NEGATIVE RESULTS
WILL APPEAR ALMOST
IMMEDIATELY.
POSITIVE RESULTS AT FIVE MINUTES.

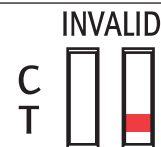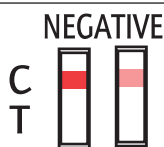

POSITIVE: A ROSE-PINK BAND IS VISIBLE IN EACH CONTROL ZONE (TOP BAND) WITH NO COLOR BAND APPEARING IN THE APPROPRIATE TEST ZONE (BOTTOM BAND).

NEGATIVE: A ROSE-PINK BAND IS VISIBLE IN EACH CONTROL ZONE (TOP BAND) AND IN THE APPROPRIATE TEST ZONE (BOTTOM BAND). A FAINT, FADED, OR BROKEN BAND IS NEGATIVE.

INVALID: A COLOR BAND IS NOT VISIBLE IN EACH OF THE CONTROL ZONES OR NO CONTROL BAND.

THIS IS A QUALITATIVE RESULT. WE RECOMMEND THAT IF YOU HAVE ANY QUESTIONS TO SEND IN FOR CONFIRMATION. ML25000.27 REV B